



1084 – Baby Bassinet

User's Manual

Index

1. Introduction	3
2. About this Product	3
3. Definitions	4
4. Safety Instructions	4
5. Environmental Specifications	6
6. Installation	7
7. Operation	8
8. Maintenance & Cleaning	9
9. Warranty & Service	12

INTRODUCTION

About this User Manual

This User Manual contains instructions for the proper and safe use of Baby Bassinet, which has to be strictly followed.

This User Manual must be read and followed by every person using or handling these baby bassinets. In case of any query, kindly contact Midmark India Pvt. Ltd.

Intended Use/ Indication of Use

Non-invasive, non-sterile devices used independently or with Medical Devices to conveniently assist the Patient / Caregiver in a normal hospital environment.

ABOUT THIS PRODUCT

- This Baby Bassinet is a non-invasive medical device used for the supportive care of infants/newborns baby and newborns as patients during treatment by providing comfort, care, and transportability of the infants.
- This Baby Bassinet is not directly used for any diagnostic, measuring, monitoring, or therapeutic purposes or to prevent or alleviate diseases.

1084 Baby Bassinet Features

 Please refer to the 1084 catalog for the salient features of these products.

Technical Data

 Please refer to the 1084 catalog for the technical data of this product.

DEFINITIONS

Operator

The operator is the person appointed by the owner (i.e. clinic, hospital management, etc.) who will be responsible for the safe operation of this baby Bassinet.

User

A user is a person who can assess the clinical condition of the baby, e.g. specialist medical personnel, doctors, nursing staff, attendants, etc. Based on their training experience or through the instructions, they are entitled to operate the Baby Bassinet as their responsibility or to carry out work on it and can recognize and control possible risks to the patient as well as the baby bassinet.

Patient

A patient is a person who is a newborn infant less than 70cm in length accommodated in the baby bassinet. Each time the Baby Bassinet is allotted to a new patient it is recommended that the patient is instructed by the operator or the user about the functions important for his/her comfort and safety.

SAFETY

In this User Manual, the following safety symbols are used



Danger: This symbol will appear wherever safety instructions are designed to protect people from physical harm. The symbol stands for imminent danger of death or serious injury.



Caution: This symbol will appear where situations are described as a bassinet, which might be dangerous, and which might inflict slight injuries.



Advice: This symbol will appear in front of additional helpful pieces of advice.



A dot in front of the text implies the action that is required to be taken.

Safety Instructions



- Ensure that the Baby Bassinet is assembled and installed following the instructions given in this user manual.
- Restrict child visitors without proper attendants. Supervision is necessary to avoid any mishap due to unintended movement of the Baby bassinet.
- Persons with reduced physical or mental abilities & and children must not be allowed to use & and handle the Baby Bassinet without proper supervision.
- Do not allow any visitor to sit on the baby bassinet, as it may cause overloading. (Refer to section- Operation)
- Do not step on the base of the bassinet.
- Do not lean or hang on the bassinet.
- Do not lift the bassinet when the babe is in the bassinet tray.
- When a cart is not moving lock the castors.
- Do not sleep the baby on the corner or the face they can suffocate.
- Accessories or any devices used along with the Baby Bassinets should be as per their intended use specified and approved by Midmark India Pvt. Ltd.
- The Baby Bassinet is always kept and used in a normal indoor hospital environment.
- Do not keep any articles below the Baby Bassinet mainframe. It can damage the linkages and screws.
- Ensure that the Baby Bassinet moves on a smooth step-less floor without any obstructions in its path, prevent any damage to the undercarriage of the Baby Bassinet.
- Keep some gap between the walls and the Baby Bassinet while in a stationary position and while moving from one place to another.
- Keep the user manual handy with the operator/user for ready reference.
- Do not use the mattress without a cover.
- While performing the inclination, do not release your hand until the desired position is achieved.
- Place the plastic basket horizontally and make sure fits snugly into the bassinet frame.

ENVIRONMENTAL SPECIFICATIONS

Usage, Storage, and Transport

Temperature	-10° C to 50°C
Relative Humidity	0% to 95% non-condensing
Atmospheric Pressure	800 to 1060
hPa Altitude	< 200Qm


SAFETY SYMBOLS

Sr.no.	Symbol	Meaning
1		Caution
2		Do not put your leg on the bassinet base

INSTALLATION

1. Unpack all the components carefully.
2. If the packing is drenched by rain, water, or other liquids, inform the matter the operator.
3. Each Baby Bassinet is supplied with the following components as per the packing list:

MS tubular frame with castors	- 1 Nos.
Pillar	- 1 Nos.
Tray support frame	- 1 Nos.
PU Foam Mattress	- 1 Nos.
Bassinet tray.	- 1 Nos.
Hardware	-1 Packet
4. Place both the tubular frames on the ground
5. Fix the MS tubular frame with the pillar by adding the bolting to it position given on the base frame and pillars shown in the picture below.
6. Fix the bassinet support frame with the pillar and check the functions.
7. Keep the bassinet plastic tray on the support frame and make sure it snugly fits in the frame without any movement.
8. Place the mattress inside the bassinet tray.
9. Notify if any damage has occurred during transportation either to the authorized agent or directly to Midmark India Pvt. Ltd.

 Installation of the Baby Bassinet should be done only by Midmark India-trained technicians.



1. Base frame
2. Bassinet tray
3. Twin-wheel castor
4. Storage tray
5. Pillar
6. Bassinet support frame

LOAD BEARING CAPACITY



 Do not exceed the safe working load mentioned above.

Operation

- Before putting the Baby Bassinet into use for the first time ensure that all the transport securing devices and packing film are removed
- Clean and disinfect the Baby Bassinet before the first use.
- The Baby Bassinet should be used on a flat floor.
- The force required to move the Baby Bassinet from one place to another with/without the patient may vary depending on the flooring conditions.
- Check if all the operations are functioning properly.
- This Baby Bassinet should be operated only by an authorized and trained person who knows its operation.
- Hold the bassinet tray while operating the gas spring till the designed position is achieved.

MAINTENANCE & CLEANING

Maintenance

Each time before putting the Baby Bassinet into service the user must check that –

- All the functions are in satisfactory working condition before the Baby Bassinet is allotted to the next patient.
- Lubrication of all the mechanical joints/pins / moving parts/tightening of the bolts should be carried out at least once in three months, to avoid unnecessary sound and wear & and tear of the Baby bassinet.
- At regular intervals carry out a visual inspection and check for any mechanical damage (e.g. loose / missing rivets, screws, cracks, etc.)
- If any damage or malfunction is observed, immediately report this matter to the operator.
- If the Baby Bassinet is not in use, it should be stored in a dry and rodent-free indoor environment on a flat surface with a gradient of not more than 5 degrees.
- If the Baby Bassinet is not in use for a long term, it is necessary to conduct a test to check for functioning and any mechanical damage that may have been caused.
- Depending on the usage, certain moving parts may require replacement due to wear & tear. We recommend timely replacement with original spares for safety and trouble-free operation.



Any kind of fire source should be prohibited near the mattress/cloth material to avoid any fire hazard as these materials do not have flame retardant properties.

Cleaning & Disinfecting

- The Baby Bassinet is not suitable for machine washing or steam cleaning.
- Do not use a water jet for cleaning.
- The Baby Bassinet cannot tolerate the influence of strong solvents, basic or alkaline liquids & or caustic agents.
- The cleaning agents must not contain any substances that bring about any change in the structure or adhesive property of the plastic.
- The Baby Bassinet should be properly cleaned and disinfected to avoid infection and stains before allotment to a new patient.
- Immobilize the Baby Bassinet & and remove all the detachable accessories and components before cleaning. Any prolonged contact of urine or acid on mild steel can corrode the surface. Always keep the surface dry.
- Wearing suitable protective clothing, clean the surface with a disposable cloth soaked in a neutral detergent and hot water. We recommend the use of approved disinfectant and cleaning agents to clean the Baby bassinet. E.a. Bacillicidal or other similar agents
- Start by cleaning the upper sections of the Baby Bassinet& and work along all horizontal surfaces. Work methodically towards the lower sections of the bassinet. Take extra care to clean areas that may trap dust or dirt.



Do not expose the Baby Bassinet to excessive moisture as corrosion may cause personal injury or equipment damage.

Disposal



Products and their defective parts especially elements made of plastic, steel, and rubber material should be disposed of following the statutory rules and regulations prevailing for such disposal. The user is advised to contact Midmark India Pvt. Ltd. for further assistance.

Routine Inspection Check List: Inspection by the User (Recommended Frequency: Every new allotment but not later than three months)

Checks	Observation	Ok	Not ok	Description of faults
Visual check of the mechanical components				
Main Frame Screws and rivets are tightened.	Damage, missing/Loose			
Bassinet Tray	Cracked/broken.			
Gas spring check	Improper working/ gas spring failure			
PU Foam Mattress / Rexene (Product Specific)	Tear, Puncture, soiled, torn			
Performance check of all mechanical Components				
Castors (Product specific)	locking, release Remove cotton thread, Hairs, etc. from the axle			
Sign	Result of inspection		Date	

Troubleshooting Guide

Problem	Possible causes	Remedy
Castors (Product specific)	No Braking, No Movement, broken	Repair / Replace Castors
The position adjustment cannot be done, or some positions cannot be locked.	Mechanics defects, moving parts can be bent	Authorized persons should change the defective parts. Do not use the bassinet till servicing!

WARRANTY & SERVICE

Midmark India Pvt. Ltd. warrants solely to the end-users of this Baby Bassinet that for a period specified in the warranty certificate, such Baby Bassinet shall perform as per its intended use. The foregoing warranties shall not apply if the Product has been subjected to: (a) physical abuse, misuse, abnormal use, or use not consistent with Midmark India Pvt. Ltd.'s published directions, (b) any modification by anyone other than Midmark India Pvt. Ltd., or third parties designated by Midmark India Pvt. Ltd., or (c) fraud, tampering, negligence or accidents, depending upon the nature of the products.

Limitation Of Liability: In no event shall Midmark India Pvt. Ltd., be liable or obligated to the end-user in any manner for any special, non-compensatory, consequential, indirect, incidental, statutory, or punitive damages of any kind. Including, without limitation. For injury or loss of life, lost profits lost sales. Lost revenue, loss of any software or data. or loss of use of any hardware, software, or data, regardless of the form of action, whether in contract, tort, negligence, strict product liability, or otherwise, even if Midmark India Pvt. Ltd., has been informed of or is aware of the possibility of any such damages in advance. Midmark India Pvt. Ltd.'s total aggregate liability under this agreement and in relation to anything which Midmark India Pvt. Ltd., has done or not done in connection with this agreement shall be limited to the price of the products purchased and paid for by the end-user during the period in which such liability arises. The limitations set forth above shall be deemed to apply to the maximum extent permitted by applicable law and notwithstanding the failure of the essential purpose of any limited remedies.

In order to maintain functional safety and right to claim under the warranty only original 'Midmark' replacements parts may be used for ordering replacement parts, customer service request and further question, please contact our technical support department.

Disclaimer

Every effort has been made to ensure that the information in this user manual is valid. Midmark India Pvt. Ltd., reserves the right to change the information in this document without prior notice."

MANUFACTURER'S ADDRESS

Corporate Office:

Midmark India Pvt. Ltd.

Art Guild House, "B" Wing, FOF, 4th, Phoenix Market City, B-16, Lal Bahadur Shastri Marg, Kurla West, Kurla, Mumbai, Maharashtra 400070

Toll Free: 1800 22 8020 | Tel: .:+91 22 4915 3000

Registered Office:

Midmark India Pvt. Ltd.

Janak House, Opp. IOC Depot, Shaikh Mistry Road, Wadala (E),
Mumbai – 400037, India

Tel: + 91 22 2413 0407

Toll-Free No.: 1800 22 8020 | Email: technicalsupport@midmark.com



MIDMARK INDIA PVT LTD.



Art Guild House, "B" Wing, FOF, 4th, Phoenix Market City, B-16, Lal Bahadur Shastri Marg, Kurla West, Kurla, Mumbai, Maharashtra 400070 | Toll Free: 1800 22 8020 | Tel: +91 22 4915 3000 | Fax: +91 22 4915 3100 www.midmark.in | india@midmark.com