

Midmark (India) Private Limited

Head Office: Art Guild House, A-Wing, Unit No. 15 & 16, Phoenix Market City, LBS Road, Kurla (W), Mumbai - 400 070
Toll Free: 1800 22 8020 | Tel: +91 22 4915 3000 | Fax: +91 22 4915 3100 | www.midmark.in | india@midmark.com
Check the nearest **Midmark India Experience Centre** at www.midmark.in/miec

We are interested in growing relationships



xxxxxxxxxxxxxxxx

Bedside Tables
User's Manual

WARRANTY & SERVICE

Midmark India Pvt. Ltd., warrants, solely to the end users of this product that for a period specified in the warranty certificate, such product shall perform as per its intended use. The foregoing warranties shall not apply if the Product has been subjected to: (a) physical abuse, misuse, abnormal use, or use not consistent with Midmark India Pvt. Ltd.'s published directions, (b) any modification by anyone other than Midmark India Pvt. Ltd., or third parties designated by Midmark India Pvt. Ltd.; or (c) fraud, tampering, negligence or accidents, depending upon the nature of the products.

Limitation of Liability: In no event shall Midmark India Pvt. Ltd., be liable or obligated to the end user in any manner for any special, non-compensatory, consequential, indirect, incidental, statutory or punitive damages of any kind, including, without limitation, for injury or loss of life, lost profits, lost sales, lost revenue, loss of any software or data, or loss of use of any hardware, software or data, regardless of the form of action, whether in contract, tort, negligence, strict product liability, or otherwise, even if Midmark India Pvt. Ltd., has been informed of or is aware of the possibility of any such damages in advance. Midmark India Pvt. Ltd.'s total aggregate liability under this agreement and in relation to anything which Midmark India Pvt. Ltd., has done or not done in connection with this agreement shall be limited to the price of the products purchased and paid for by the end user during the period in which such liability arises. The limitations set forth above shall be deemed to apply to the maximum extent permitted by applicable law and notwithstanding the failure of the essential purpose of any limited remedies.

In order to maintain functional safety and right to claim under the warranty only original replacements parts may be used. For ordering replacement parts, customer service request and further question, please contact our technical support department.

DISCLAIMER

Every effort has been made to ensure that the information in this user manual is valid. Midmark India Pvt. Ltd., reserves the right to change the information in this document without prior notice.

MANUFACTURER'S ADDRESS

Corporate Office:

Midmark India Pvt. Ltd.
Art Guild House, A-Wing, Unit No. 15 & 16, Phoenix Market City, LBS Road, Kurla (W), Mumbai - 400 070
Toll Free: 1800 22 8020 | Tel: +91 22 4915 3000 | Fax: +91 22 4915 3100
www.midmark.in | india@midmark.com

Registered Office:

Midmark India Pvt. Ltd.
Janak House, Opp. Indian Oil Corp. Depot, Sheikh Misry Road, Wadala (E), Mumbai - 400 037.
Tel: +91 22 2412 0171/2413 0407 | Fax: +91 22 2413 9870

Toll Free No.: 1800 22 8020, Email: technicalsupport@janakhealthcare.com



1. Introduction
2. About this Product
3. Definitions
4. Safety
5. Environmental Specifications
6. Installation
7. Operation
8. Maintenance & Cleaning
9. Warranty & Service

Routine Inspection Check List: Inspection By The User

(Recommended Frequency: Every new allotment but not later than one month)

Checks	Observation	OK	Not OK	Description of faults
Visual check of the mechanical components & Membrane Top				
Screws and rivets are tightened	Missing / loose			
Performance check of all mechanical components				
Castors	Smooth movement. Remove cotton thread, hairs etc. from axle			
Accessories (Product Specific)	Fastening, damage, missing			
Sign	Result of inspection		Date	

Trouble Shooting Guide

Problem	Possible Causes	Remedy
Gas Spring (for Ax1016)	Not Working / Oil Leakage	Replace Gas Spring
Castors	Breakages or Improper Movement	Repair / Replace Castors

MAINTENANCE & CLEANING

Maintenance

Each time before putting the Trolley into service the user must check for -

- Lubrication of all the mechanical joints / pins / moving parts / tightening of the bolts are done properly, to avoid unnecessary play, sound and wear & tear of the Bedside Table
- Visual inspection for any mechanical damage (e.g. loose /missing screws, etc.)
- If any damage or malfunction is observed, immediately report this matter to the operator
- If the Bedside Tables are not in use, it should be stored in a dry and rodent free indoor environment on a flat surface with gradient not more than 5 degrees
- If the Bedside Tables are not in use for long term, it is necessary to conduct a test to check for functioning and any mechanical damage that may have been caused
- Depending on the usage, certain moving parts may require replacement due to wear & tear. We recommend timely replacement with original spares for safety and trouble free operation

Cleaning & Disinfecting

- The Bedside Tables are not suitable for machine wash or steam cleaning
- Do not use water jet for cleaning
- The Bedside Tables cannot tolerate the influence of strong solvent, basic or alkaline liquids & caustic agents
- The cleaning agents must not contain any substances which bring about any change in the structure or adhesive property of the plastic
- The Bedside Tables should be properly cleaned with a damp cloth and disinfected at regular intervals to avoid infection and stains
- We recommend use of approved disinfectant and cleaning agents to clean the Bedside Tables. Take extra care to clean areas that may trap dust or dirt. All detachable accessories and its component can be cleaned separately



Do not expose the Bedside Tables to excessive moisture as corrosion may cause personal injury or equipment damage.

Disposal:



Product and its defective parts especially elements made of plastic, steel, rubber and electrical material should be disposed off in accordance with the statutory rules and regulations prevailing for such disposal. The user is advised to contact Midmark India Pvt. Ltd., for further assistance.

INTRODUCTION

About this User Manual

This User Manual contains instructions for the proper and safe use of Bedside Tables, which has to be strictly followed.

This User Manual has to be read and followed by every person using or handling these Bedside Tables. In case of any query, kindly contact Midmark India Pvt. Ltd.

ABOUT THIS PRODUCT

The Bedside Tables are manufactured and supplied in non-sterile condition for use in normal hospital environment.

These Bedside Tables are not used for any diagnostic, measuring, monitoring or therapeutic purposes or prevent nor alleviate diseases.

Overbed / Bedside tables are generally used by the patient to hold food while eating. These are available with fixed height as well as height adjustment which are manually operated.

Bedside Tables Features

Ax1016A - Please refer product catalogue WCS - A for salient features of these products.

Ax1012, Ax1011, Ax1014 - Please refer product catalogue WCS -B for salient features of these products.

Technical Data

Ax1016 - Please refer product catalogue WCS -A for Technical Data of these products.

Ax1012A, Ax 1011, Ax1014 - Please refer product catalogue WCS -B for Technical Data of these products.

Wardcare Accessories and Peripherals – Variants



Ax1016A



Ax1012A



Ax1011



Ax1014

Ax1011





1. Attach both the Rectangular frame with each other.
2. Fix the Laminated top on the above with the hardware's provided.



With due consideration to environmental limitations, materials used to make the device are not affected by contact with any bio chemicals or bio waste. When used as per its intended purpose, all materials used in the product and accessories do not facilitate any breeding or growth of bio contaminants on their own.

Load Bearing Capacity of Bedside Tables

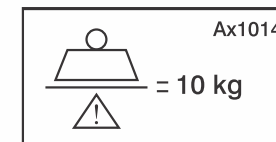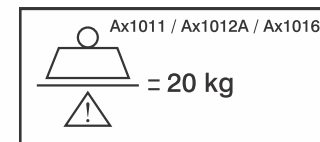

Do not load the Bedside Tables more than the Safe Working Load as mentioned above.

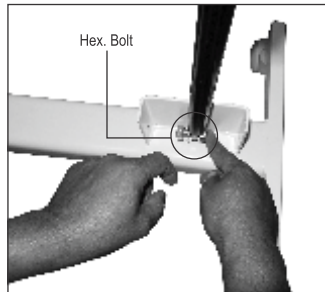


Fig 1

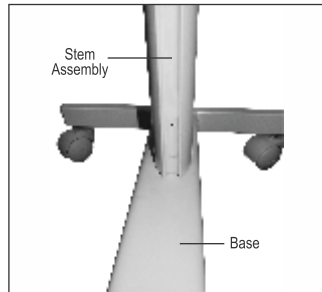


Fig 2

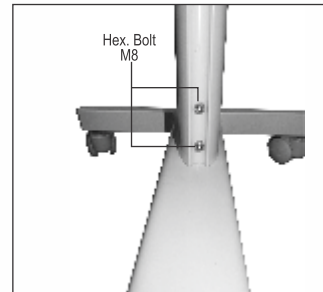


Fig 3

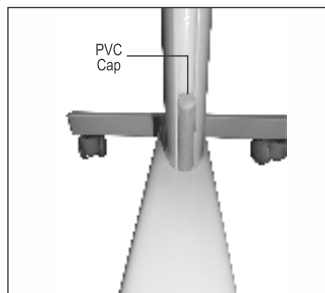


Fig 4

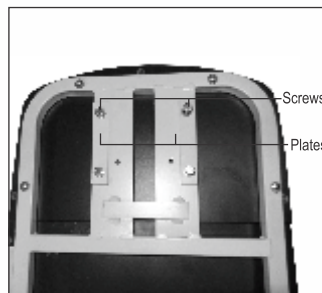


Fig 5

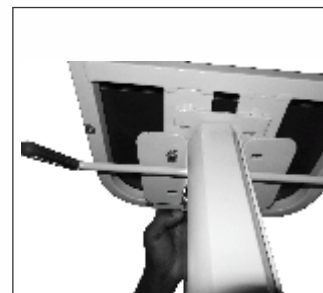


Fig 6

1. Place the base frame on the ground.
2. Fix the gas spring on the base frame with the hex bolt (Fig 1).
3. Place the stem assembly on the base and slide the aluminium extrusion stem (Fig 2) and fix it with hex bolt M8 (Fig 3).
4. Cover the bolts with the PVC Caps provided on both the sides (Fig 4).
5. Place the top (Fig 5) upside down on the flat surface. Put some packing sheet below the top to prevent any damage.
6. Remove the screws and plates fixed on the top (Fig 5).
7. Turn the assembly of stem with base frame upside down and fix the top with bracket using screw pan head 3/16" (Fig 6).
8. Turn the assembled table in upright position with castor on the ground.

DEFINITIONS

Operator

Operator is the person appointed by the owner (i.e. clinic, hospital management, etc.) who will be responsible for the safe operation of these Bedside Tables.

User

User is a person who is capable of assessing clinical condition of patient e.g. specialist medical personnel, doctors, nursing staff, attendants etc. On the basis of their training experience or through the instructions, they are entitled to use these Bedside Tables on their own responsibility or to carry out work on it and are able to recognize and control possible risks to the patient as well as the Bedside Tables.

Patient

The patient is a person who is ill, infirm, disabled or in need of medical assistance.

SAFETY

In this User Manual the following safety symbols are used.



Danger: This symbol will appear wherever safety instructions are designed to protect people from physical harm. The symbol stands for imminent danger of death or serious injury.



Caution: This symbol will appear where situations are described, which might be dangerous, and which might inflict slight injuries.



Advice: This symbol will appear in front of additional helpful pieces of advice.

- A dot in front of the text implies the action that is required to be taken.

Safety Instructions

- Ensure that the Bedside Tables are assembled and installed in accordance with the instructions given in this User Manual (wherever required)
- Restrict child visitors without proper attendants. Supervision is necessary to avoid any mishap due to unintended movement of the Bedside Tables
- Ensure that the Bedside Tables is moved on a smooth stepless floor without any obstructions in its path, to prevent any inconvenience to User and damage to the Bedside Tables
- Keep some gap between walls and Bedside Tables while in stationary position and also while moving from one place to another
- Keep the User Manual handy with the Operator/User for ready reference

ENVIRONMENTAL SPECIFICATIONS

Usage, Storage and Transport

Temperature	-10°C to 50°C
Relative Humidity	0% to 95% non condensing
Atmospheric Pressure	800 to 1060 hPa
Altitude	< 2000m

Installation and Operation

Ax1016A, Ax1012A, Ax1011 & Ax1014

1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.
3. Each Product is supplied with the following components as per packing list :

Base Frame (Ax1012 & Ax1016)	- 01 No.
Telescopic Stem with Gas Spring (Ax1016)	- 01 No.
Membrane Press top with Ratchet rod (Ax1016)	- 01 No.
Telescopic Stem with bolts (Ax1012)	- 01 No.
Laminated Top (Ax1012)	- 01 No.
M.S. rectangular Tubes mounted with 4 Twin wheels LH & RH (Ax1011)	- 1 Pair
Hardware	- 1 Packet

Notify if any damage has occurred during transportation either to the authorized agent or directly to Midmark India Pvt. Ltd.

Installation of the Bedside Tables should be done only by Janak trained technicians.

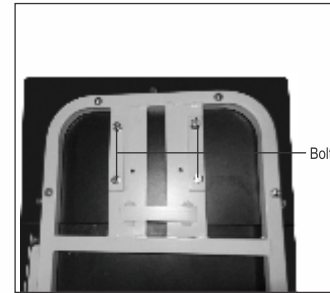


Fig 1

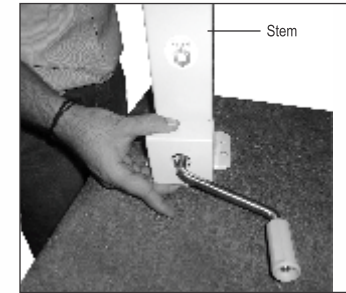


Fig 2

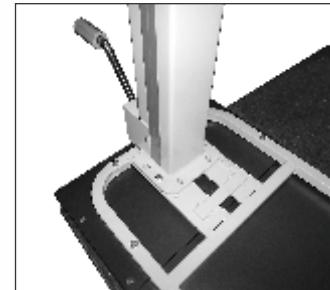


Fig 3

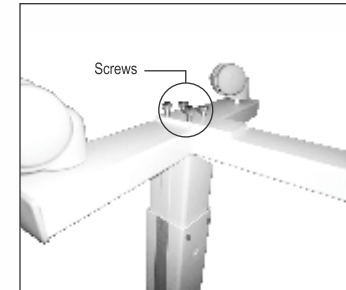


Fig 4

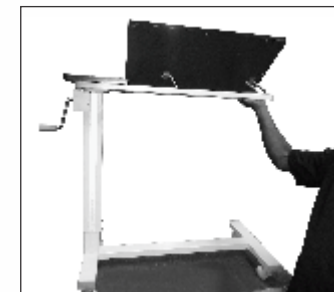


Fig 5

1. Place the top upside down on the flat surface (Fig 1) and remove the bolt from the top.
2. Place the stem (Fig 2) on the top as shown in (Fig 3) and fasten it with top using bolts.
3. Hold the base frame (Fig 4) upside down on base of the stem and fasten it using the bolts.
4. Turn the assembled table in upright position with the castors (Fig 5) on the ground.