



Midmark (India) Private Limited

Head Office: Art Guild House, A-Wing, Unit No. 15 & 16, Phoenix Market City,
LBS Road, Kurla (W), Mumbai - 400 070
Toll Free: 1800 22 8020 | Tel: +91 22 4915 3000 | Fax: +91 22 4915 3100
www.midmark.in | india@midmark.com
Check the nearest **Midmark India Experience Centre** at www.midmark.in/miec

We are interested in growing relationships



BEDSIDE LOCKERS

User's Manual



Index



WARRANTY & SERVICE

Midmark India Pvt. Ltd., warrants, solely to the end users of this product that for a period specified in the warranty certificate, such product shall perform as per its intended use. The foregoing warranties shall not apply if the Product has been subjected to: (a) physical abuse, misuse, abnormal use, or use not consistent with Midmark India Pvt. Ltd.'s published directions, (b) any modification by anyone other than Midmark India Pvt. Ltd. or third parties designated by Midmark India Pvt. Ltd., or (c) fraud, tampering, negligence or accidents, depending upon the nature of the products.

Limitation Of Liability: In no event shall Midmark India Pvt. Ltd., be liable or obligated to the end user in any manner for any special, non-compensatory, consequential, indirect, incidental, statutory or punitive damages of any kind, including, without limitation, for injury or loss of life, lost profits, lost sales, lost revenue, loss of any software or data, or loss of use of any hardware, software or data, regardless of the form of action, whether in contract, tort, negligence, strict product liability, or otherwise, even if Midmark India Pvt. Ltd. has been informed of or is aware of the possibility of any such damages in advance. Midmark India Pvt. Ltd.'s total aggregate liability under this agreement and in relation to anything which Midmark India Pvt. Ltd., has done or not done in connection with this agreement shall be limited to the price of the products purchased and paid for by the end user during the period in which such liability arises. The limitations set forth above shall be deemed to apply to the maximum extent permitted by applicable law and notwithstanding the failure of the essential purpose of any limited remedies.

In order to maintain functional safety and right to claim under the warranty only original replacement parts may be used. For ordering replacement parts, customer service request and further question, please contact our technical support department.

DISCLAIMER

"Every effort has been made to ensure that the information in this user manual is valid. Midmark India Pvt. Ltd., reserves the right to change the information in this document without prior notice."

MANUFACTURER'S ADDRESS

Corporate Office:

Midmark India Pvt. Ltd.
Art Guild House, A-Wing, Unit No. 15 & 16, Phoenix Market City, LBS Road, Kurla (W), Mumbai - 400 070
Toll Free: 1800 22 8020 | Tel: +91 22 4915 3000 | Fax: +91 22 4915 3100
www.midmark.in | india@midmark.com

Registered Office:

Midmark India Pvt. Ltd.
Janak House, Opp. Indian Oil Corp. Depot, Sheikh Misry Road, Wadala (E), Mumbai - 400 037.
Tel: +91 22 2412 0171/2413 0407 | Fax: +91 22 2413 9870

Toll Free No.: 1800 22 8020, Email: technicalsupport@midmark.com

1. Introduction
2. About these Product
3. Definitions
4. Safety
5. Environmental Specifications
6. Installation
7. Operation
8. Maintenance & Cleaning
9. Warranty & Service



Routine Inspection Check List: Inspection by the User

(Recommended Frequency: Every new allotment but not later than one month)

| Checks | Observation | OK | Not OK | Description of faults |
|---|--|----|--------|-----------------------|
| Visual check of the mechanical components & Laminated Top, Front Door & Drawer Top | | | | |
| Screws tightened | Missing / loose | | | |
| Performance check of all mechanical components | | | | |
| Castors | Remove cotton thread, hairs etc. from axle | | | |
| Sign | Result of inspection | | Date | |

Trouble Shooting Guide

| Problem | Possible Causes | Remedy |
|----------------|--------------------------------|--------------------------|
| Castors | Breakages or improper movement | Repair / Replace Castors |

MAINTENANCE & CLEANING

Maintenance

While using the Bedside Locker, User must check for the following:

- The Drawers, Doors, and locking function are working properly
- The Bedside Locker is not damaged, dirty, or in wet condition

Cleaning & Disinfecting

- The Bedside Lockers are not suitable for machine wash or steam cleaning
- Do not use water jet for cleaning
- The Bedside Lockers cannot tolerate the influence of strong solvent, basic or alkaline liquids & caustic agents
- The cleaning agents must not contain any substances which bring about any change in the structure or adhesive property of the plastic
- The Bedside Lockers should be properly cleaned and disinfected at regular intervals to avoid infection and stains
- We recommend use of approved disinfectant and cleaning agents to clean the Bedside Lockers. Take extra care to clean areas that may trap dust or dirt. All detachable accessories and its component can be cleaned separately



Do not expose the Bedside Lockers to excessive moisture as corrosion may cause personal injury or equipment damage.

Disposal:



Product and its defective parts especially elements made of plastic, steel and rubber material should be disposed off in accordance with the statutory rules and regulations prevailing for such disposal. The user is advised to contact Midmark India Pvt. Ltd., for further assistance.

INTRODUCTION

About this User Manual

This User Manual contains instructions for the proper and safe use of Bedside Lockers, which has to be strictly followed.

This User Manual has to be read and followed by every person using or handling these Bedside Lockers. In case of any query, kindly contact Midmark India Pvt. Ltd.,

About these Products

The Bedside Lockers are manufactured and supplied in non-sterile condition for use in normal hospital environment

These Bedside Lockers are not used for any diagnostic, measuring, monitoring or therapeutic purposes or prevent nor alleviate diseases

Bedside Lockers are used by patients for storing the belongings, medicines etc.

Bedside Lockers Features

Please refer product catalogue WCS -A for salient features of these products.

Technical Data

Please refer product catalogue WCS -A for Technical Data of these products.

Bedside Lockers - Variants



Ax1009 / Ax1009B



Ax1006 / Ax1006A



Ax1001

DEFINITIONS

Operator

Operator is the person appointed by the owner (i.e. clinic, hospital management, etc.) who will be responsible for the safe operation of these Bedside Lockers.

User

User is a person who is capable of assessing clinical condition of patient e.g. specialist medical personnel, doctors, nursing staff, attendants, etc. On the basis of their training experience or through the instructions, they are entitled to use these Bedside Lockers on their own responsibility or to carry out work on it and are able to recognize and control possible risks to the patient as well as the Bedside Lockers.

Patient

The patient is a person who is ill, infirm, disabled or in need of medical assistance.

SAFETY

In this User Manual the following safety symbols are used

 **Danger:** This symbol will appear wherever safety instructions are designed to protect people from physical harm. The symbol stands for imminent danger of death or serious injury.

 **Caution:** This symbol will appear where situations are described, which might be dangerous, and which might inflict slight injuries.

 **Advice:** This symbol will appear in front of additional helpful pieces of advice.

- A dot in front of the text implies the action that is required to be taken.

Safety Instructions

- Ensure that the Bedside Lockers are assembled and installed in accordance with the instructions given in this User Manual (wherever required)
- Restrict child visitors without proper attendants. Supervision is necessary to avoid any mishap due to unintended movement of the Bedside Lockers
- Ensure that the Bedside Lockers are moved on a smooth step-less floor without any obstructions in its path, to prevent any inconvenience to User and damage to the Bedside Lockers
- Keep some gap between walls and Bedside Lockers while in stationary position and also while moving from one place to another
- Keep the User Manual handy with the Operator/User for ready reference

ENVIRONMENTAL SPECIFICATIONS

Usage, Storage and Transport


| | |
|----------------------|--------------------------|
| Temperature | -10°C to 50°C |
| Relative Humidity | 0% to 95% non condensing |
| Atmospheric Pressure | 800 to 1060 hPa |
| Altitude | < 2000m |

Installation and Operation

Ax1009, Ax1009B, Ax1006 & Ax1006A Bedside Lockers come in ready to use condition from the factory.

1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.
3. Each Ax1001 Bedside Locker is supplied with the following components as per packing list:


| | |
|-----------------------------|------------|
| M.S Tubes with Rubber Shoes | - 4 Nos. |
| Locker Box with door | -1 No. |
| S.S. Top | -1 No. |
| Hardware | - 1 Packet |

 Notify if any damage has occurred during transportation either to the authorized agent or directly to Midmark India Pvt. Ltd.

 Installation of the Bedside Locker should be done only by Janak trained technicians.

Ax1001 Installation

1. Fix the M.S. Tubes on the Locker box by using the hardware provided.
2. Upright the Locker & fix the S.S. Top with the screws provided.

 With due consideration to environmental limitations, materials used to make the device are not affected by contact with any bio chemicals or bio waste. When used as per its intended purpose, all materials used in the product and accessories do not facilitate any breeding or growth of bio contaminants on their own.