



Stanchion

User's Manual

Index

| | |
|---------------------------------|----|
| 1. Introduction | 3 |
| 2. About this Product | 3 |
| 3. Definitions | 4 |
| 4. Safety | 4 |
| 5. Electrical Specifications | 5 |
| 6. Environmental Specifications | 5 |
| 7. Installation | 7 |
| 8. Add-On Accessories | 8 |
| 9. Operation | 8 |
| 10. Maintenance & Cleaning | 9 |
| 11. Warranty & Service | 13 |

INTRODUCTION

About this User Manual


This user manual contains instructions for the proper and safe use of the Stanchion Motorized Beds and for patient & user safety and comfort, which has to be strictly followed.

This User Manual has to be read and followed by every person using or handling this Bed. In case of any query, Kindly contact Midmark India Pvt. Ltd.

ABOUT THIS PRODUCT


- These Motorized Beds are non-invasive medical devices used for resting and treatment of the patients by medical / paramedical / hospital personnel to help and support the patient in diagnosis, prevention, monitoring, treatment or alleviation of the disease.
- These advanced Motorized Beds have patient support and patient rest facility which help medical personnel in carrying out their medical procedures comfortably, minimizing the risks while moving and handling the patient.
- These advanced Motorized Beds are manufactured and supplied in non-sterile condition for use in normal hospital environment.
- These advanced Motorized Beds are electrically operated for certain profiling movements to provide clinical positions with the help of patient and caregiver friendly embedded panels and digital Attendant control. These Beds are supported with a Battery Backup.
- These advanced Motorized Beds are not diagnostic, measuring, monitoring or a therapeutic device or we say that it can prevent or alleviate disease.

Stanchion Bed Features




Please refer to the Stanchion catalogue for the salient features of this product.

Technical Data



Please refer to the Stanchion catalogue for the technical data of this product.

Add – On Accessories



Please refer to the Stanchion catalogue for various accessories available and compatible with this product.

DEFINITIONS

Operator

The operator is the person appointed by the owner (i.e. clinic, hospital management, etc.) who will be responsible for the safe operation of this Bed.

User

A user is a person who is capable of assessing the clinical condition of the patient e.g. specialist medical personnel, doctors, nursing staff, attendants, etc. Based on their training experience or through the instructions, they are entitled to operate the Bed on their responsibility or to carry out work on it and can recognize and control possible risks to the patient as well as the Bed.

Patient

A patient is a person who is ill, infirm, disabled, or in need of care & occupying this Bed. Each time the Bed is allotted to a new patient it is recommended that the patient is instructed by the operator or the user about the functions important for his/her comfort and safety.

SAFETY

In this User Manual, the following safety symbols are used



Danger: This symbol will appear wherever safety instructions are designed to protect people from physical harm. The symbol stands for imminent danger of death or serious injury.



Caution: This symbol will appear where situations are described, which might be dangerous, and which might inflict slight injuries.



Advice: This symbol will appear in front of additional helpful pieces of advice.



A dot in front of the text implies the action that is required to be taken.

Safety Instructions

- Always use Midmark (India) accessories that have been designed and approved for use with the Bed.
- Ensure that the Bed is assembled and installed in accordance with the instructions given in this User Manual.
- Restrict child visitors without proper attendants. Supervision is necessary to avoid any mishap due to the unintended movement of the Bed.
- Persons with reduced physical or mental abilities & children must not be allowed to use & handle the Bed without proper supervision.
- Leave the bed in the lowest position when the patient is unattended. Leaving the bed in a raised position could increase the chance of patient falls and injury.
- Weighing scale function may get damaged due to potential interference between castors and base frame. When bed in lowest height, castors tending outwards and Reverse Trendelenburg position, side rails if lowered can interfere with base frame damaging the load cells placed in base.
- Do not allow any visitor to sit on the Bed, as it may cause overloading. (Refer Section-Operation)
- Accessories or any devices used along with the Bed should be as per its intended use specified and approved by Midmark (India).
- The Bed should always be kept and used in normal indoor hospital environment.
- The Bed is not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide. It may damage the Electro-Mechanical Control System.
- Always use a mattress of the correct size and type. Before operating the Bed, make sure that the patient is safely positioned to avoid entrapment or imbalance.
- Do not keep any articles on the Base Frame Cover of the Bed. It can damage the linkages and electrical components.
- Keep the User Manual handy with the Operator/User for ready reference.



Keep some gap between walls and bed while in a stationary position and also while moving from one place to another




Use of water on mattress is prohibited.

Electrical Safety Points

When the Bed is connected to 230V / 15A (or as per country specific) main supply-

Connecting more than one plug to the three pin mains socket can cause electrical overload and damage the control system of the Bed.

- AC power supply exceeding 230V or sudden spikes or lack of proper earthing can be hazardous to User / Patient, as well as can cause damage to the Bed and property.
- Ensure stabilized and properly earthed input power supply for the equipment. Also, ensure polarity of the mains socket is as per Standard and matches with the mains plug.
- Inspect all the wires and cables for any cuts, cracks, and damages at regular intervals.

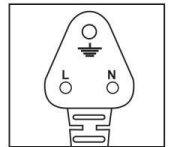

ELECTRICAL SPECIFICATIONS

| | |
|---|---|
| Mains power supply | Nominal 230 VAC (150 VAC to 300VAC without battery) (90 VAC to 300VAC with battery) |
| Frequency | 50Hz + /- 5Hz |
| Protection Voltage | 300 V to 450 V for 230 VAC system |
| Mode of operation | 10% (2min. in use /18 min. not in use) |
| Power consumption idle mode without battery | 5W |
| Power consumption idle mode with battery | 10W |
| Power consumption at maximum load | 300 Wmax. |
| Mains Fuse DC Bus Fuse | 2A fast blow for mains (external) 10A fast blow fuse (internal) |
| Protection Class | Class I |
| Degree of Shock Protection | Type B |

Inbuilt Battery Charger

Battery Charger (Inbuilt)

- Suitable to charge 1.3AhX 2 batteries
- Reverse polarity protection
- Spark free connection
- Short circuit protected output.
- Very minimal battery consumption at idle state, to enhance standby time.
- Battery overcharging protection.
- Battery low or dip discharge protection.

ENVIRONMENTAL SPECIFICATIONS

Usage, Storage and Transport

| | |
|----------------------|--------------------------|
| Temperature | -10° C to 50°C |
| Relative Humidity | 0% to 95% non-condensing |
| Atmospheric Pressure | 800 to 1060 hPa |
| Altitude | < 200Qm |

INSTALLATION



1. Unpack all the components carefully.
2. If the packing is drenched by rain, water or other liquids, inform this matter to the operator.
3. Each Bed is supplied with the following components as per

packing list:-

| | |
|---|-----------|
| Bed with embedded panel equipped side rails | -1 No. |
| Rubber Buffer with bush | - 4 Nos. |
| Polymer Molded Head and Foot Board | -1Pair |
| Miscellaneous items | -1 Packet |

4. Prior to connecting the bed to the mains, get the input voltage & Earthing tested through a qualified electrician. It is necessary to ensure that all electrical connections are proper and tightened.
5. It is advisable to charge the battery for 24 hours to get the best performance when plugged in for first time
6. For using the Embedded panel and ACP (Product Specific) check all the functions by pressing the respective buttons.
7. Demonstrate the functions of the Bed to the User / Operator.
8. Notify if any damage has occurred during transportation either to the authorized agent or directly to Midmark (India) Pvt. Ltd.
9. Installation of the Bed should be done only by Midmark trained technicians.



Notify if any damage has occurred during transportation either to the authorized agent or directly to Midmark India Pvt. Ltd.



Installation of the Bed should be done only by Midmark India trained technicians

1. Head & Foot Boards:



- Fix Head and Foot Board by extending the equal forces with both the hands in the bracket.

2. Five Section Metal (P.P.) Top:

- Place the individual Polypropylene sections of Backrest (Upper), Backrest (Lower), Middle, Upper Leg rest and Lower Leg rest on their respective positions on the Bed top frame as shown in the figure above.



3. Urine Bag Holder


Provision to hold the urine bag on the bed for immovable/critical patients



4. Premium single section mattress.



5. Buffers

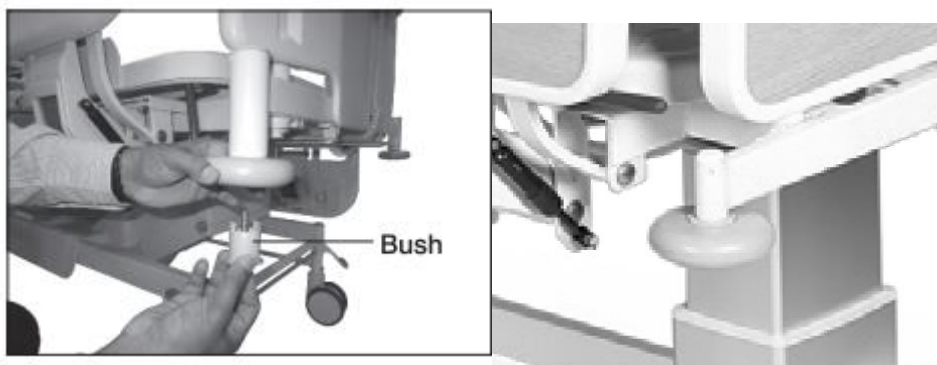


- Fix the rubber buffer in the buffer holder with bush and bolt at all 4 corners

ADD ON Accessories

a. Heavy Duty I.V. Pole (2 Hook) / (4 Hook)




Heavy Duty I.V. Pole (0164)

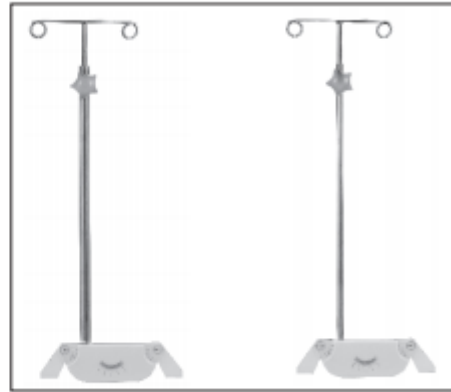


Utility I.V. Pole
(0164A)

Telescopic I.V. Pole
(0154A)

Used to hang IV bags for infusion of liquid substances directly into the vein.

 Do not overload the I.V. Pole with more than 20 kgs. Equally distributed on all hooks.

B. S.S. Lifting Pole

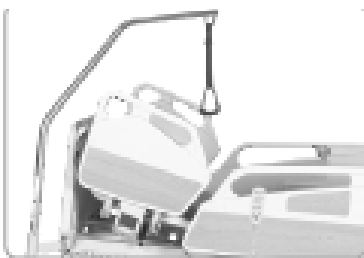

Helps the patient lying on the Bed to lift themselves up on their own, by holding the hand grip.

C. Moulded Chart Holder



Used for keeping medical reports and charts.

D. Traction pulley attachment for trauma cases.-



Used in orthopaedic treatment for physical therapy

E. Oxygen Cylinder Cage Vertical Type



Used to hold the oxygen cylinder when transferring the critical patient.

F. Four Section Mattress




G. Mattress for Bed extension




Bed Extension Mattress

H. Side Railings



 Ensure that the Accessories are properly secured and not overloaded to avoid any personal injury or equipment damage.

 Use of Bed accessories in accordance with the manufacturer's instructions for patient safety and comfort. Use recommended spares and accessories of suitable design and manufactured by "Midmark India Pvt. Ltd." to eliminate hazards.

****Available For Stanchion Bed Variants****

X-Ray Permeable Backrest with Cassette Holder

- The Backrest Support Top in this option is made of high pressure laminated sheet which is X-Ray permeable.
- This facility is provided specially for patients who are seriously ill or too weak to stand unsupported, and need the X-Ray imaging without being moved out of the bed.
- Beds having with X-Ray Permeable Backrest come factory fitted to the bed, and do not need any special installation.



Fig. 1



Fig. 2



Fig. 3



Fig. 4



Fig. 5

Procedure to use X-Ray Backrest

- Raise the Back rest to a convenient position as shown in Fig. 1&2.
- Unscrew both the knobs provided at the sides of the Cassette holder as shown in Fig.3.
- Slide the Cassette holder upwards and outwards from its guides as shown in Fig.4.
- Place the X-Ray plate on the Cassette holder to its desired position for X-Ray imaging.
- Lock the plate in position by adjusting the levers shown in Fig.5.
- Insert back the Cassette holder with Plate in its guides.
- Tighten both the knobs provided at sides.
- Adjust the Backrest to facilitate X-Ray imaging.



Only authorized and qualified persons designated by the Operator and having proper training should be allowed to prepare the bed with patient for X-Ray imaging.

ELECTRICAL CONTROL SYSTEM

The electrical control System of this Bed consists of:


- Four Actuator to facilitate the movement of Back Section, Upper Leg Section, Height, Trendelenburg / Reverse Trendelenburg Tilt.
- User Friendly Handset is provided for achieving various clinical positions.
- An Attendant Control Panel-ACP (Product Specific) is provided for achieving various clinical positions and locking any function in the Handset to avoid any unintentional misuse of the Bed by User.
- A control unit is located below the main frame of the Bed to which all the other electrical components are connected through a Junction Box. A battery box containing two batteries (for power backup in an emergency and during the power failures) is provided below the main frame of the Bed. Night Lamps provided below the main frame of the Bed which works on mains supply only.

OPERATION

- Before putting the Bed into use for the first time ensure that all the transport securing devices and packing film are removed.
- Clean and disinfect the Bed prior to the first time use.
- The Bed should be used on a flat floor.
- The force required to move the Bed from one place to another with/without the patient, may vary depending on the flooring conditions.
- Check if all the operations are functioning properly.
- This Bed should be operated only by an authorized and trained person who has knowledge of its operation.



Do not stretch/ extend the power supply Cord from its plug.

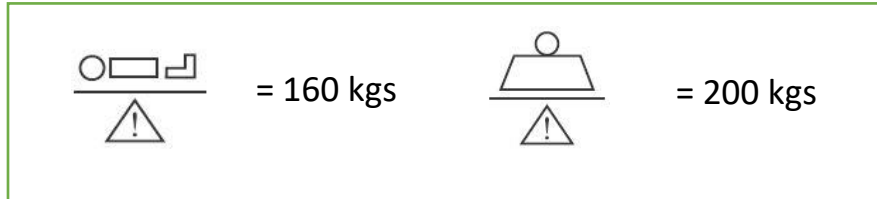

Do not load the Bed more than the Safe Working Load as mentioned above.

MECHANICAL ADJUSTMENTS

1. Operation of the Central Locking

The Central Locking Castors are provided with a lever which has got three positions.



When pushing the foot rod downwards, all Castors completely get locked and the Bed does not move at all.




When pushing the foot rod to its middle position, all the Castors are free and can be moved to any position.



When pushing the foot rod to its upper most position, the steering Castors gets locked to swivel which help the Bed to move in a straight direction.

2. Lower Leg Section

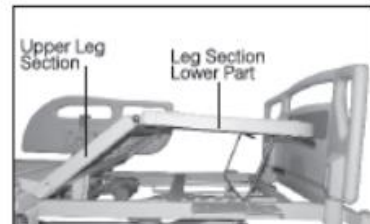

Lower Leg Section

Lower Leg Section gets raised simultaneously when raising the Upper Leg Section through electrical control. To attain the Calf Elevation position, the Lower Leg Section can be adjusted manually.

To Raise Leg Section (Lower part) manually –

- Firmly hold and rat the Leg Section (Lower part) with one hand.
- With the other hand hold the Ratchet Handle to the required inclination.
- Place the Ratchet Handle in the desired Ratchet Notch to lock the position.
- Make the patient comfortable by readjusting his position on the Bed.



To Lower Leg Section (Lower part) manually –



- Firmly hold the Leg Section (Lower part) with one hand.
- Pullout the ratchet handle from the ratchet notch to unlock the position.
- Carefully place the Leg Section (Lower Part) on the Main frame.



Always observe the correct adjustment sequence.

Care should be taken to avoid fingers and hand getting entrapped between the Ratchet and the top frame.

3. CPR Release Handle

Emergency lowering of the Back Section using CPR Release Handle

The CPR Release Handle is to be used to bring the Back Rest to its flat position in case of emergency or power failure.



CPR RELEASE HANDLE



Procedure for Operating CPR lever manually:

- Hold the Back Section with one hand while operating CPR Release Handle.
- With one hand pull the operating lever (CPR Release Handle) so that the Back Section can be lowered. The Back Section Actuator is locked in this process.
- The Back Section does not stay in its flat position until the CPR Handle is released.
- Once the flat position is achieved. Release the CPR Release Handle.
- The Back Section can be operated with the help of control elements.

Since the Back Section is designed for motorized adjustment, manual CPR should be used only in case of emergency. Continuous manual adjustment may damage the Actuator. As the Backrest can fall quickly, keep hands clear to avoid trapping.

4. Head & Foot Boards

- The Head and Foot Board are located at the head and foot end of the Bed frame.
- The Head and Foot Board can be removed or attached in a single step by extending equal force with both the hands in the bracket by ensuring the proper matching of the brackets with board holder fitted in the main frame.
- The Head and Foot Boards can be removed for ease of access to the patient from head and foot side.



Do not hang any heavy items on the Head and Foot Boards.

To use the Safety Side Railings

- Immobilize the Bed.
- To raise the Railing, pull it to the top position until it gets locked.
- To release / lower the Safety Side Railing, ensure that there are no obstacles in the path of the railing.
- Lift the lever and push the Safety Side Railing down to its end position.
- Always keep the Safety Side Railing in raised condition when moving the patient with the Bed. Do not use the Safety Side Railing to lift or move the Bed.
- Safety Side Railings must only be used with a mattress of the correct size and which is approved for use with the Bed.
- To prevent the patient from falling, ensure that the Safety Side Railings are in raised

position.

- The clinically qualified person responsible should consider the age, size and condition of the patient before allowing the use of the Safety Side Railings.
- Other manufacturer’s Safety Side Railings are not compatible with these Beds.
- Please take care of fingers and hands while operating the Safety Side Railings.



Although the Safety Side Railings have been designed to reduce the risk of patient injury, the potential exists for patient entrapment, particularly in agitated or disoriented patients, patients

who have cognitive disability or have uncontrolled body movement as well as patients who lack the physical strength to extract themselves, if they get entrapped.



Users should carefully evaluate the need for Safety Side Railings and periodically check patients in accordance with facility protocols for safe positioning.















The Safety Side Railings should always be in raised position when attaining lowest height position or when attaining Trendelenburg / Reverse Trendelenburg Tilt positions.




If these safety measures are not observed, injuries can be caused to hands, knees, fingers, feet, legs or other body parts of patient due to unintended and unintentional entrapment. In children or weak patients who are less than 146 cms tall, non-observance of these guidelines may even lead to death.

Electrical Adjustments


- 1. Power ON/OFF Button:** Press the power button to put the system ON. With this button the system is alive. Press again the power button to put the system OFF. Once you OFF the system, side rail Embedded system will also turn OFF.
- 2. Feature Lock Button:** Press the feature lock button to lock the required clinical positions at any given time. Once you lock any clinical positions and if you try to change it, LED light placed at top right of the lock feature button will blink indicating that feature is locked by user. Once feature is locked, same feature cannot be accessed from embedded system in side rails. Press again the feature lock button to unlock the feature.

3. **Emergency Head down button:** Single continuous press the button to lower the head section of the bed for emergency cases.
4. **CPR:** Single continuous press the button will enable the bed to become flat and height of the bed goes to its lowest for nursing staff to perform emergency activities.
5. **Back Section Buttons:** Press  button to raise the section. Press button  to lower the backrest section.
6. **Upper Leg Section  Buttons:** Press button to raise the upper leg section. Press  button to lower the upper leg section.
7. **Auto-Contour Buttons:** Press  button to raise back section and upper leg section Simultaneously. Press  button to lower back section and upper leg section
8. **Bed Height Buttons:** Press  button to raise the height of bed. Press  button to lower the height of bed.
9. **Trendelenburg/Reverse-Trendelenburg Buttons:** Press  button to lower the leg section (Reverse Trendelenburg). Press  button to lower the head section.
10. **Sequential Chair Buttons:** Press  button to attain the chair position in sequential mode. Press  button to attain the flat position of bed sequential.

Using Embedded panels:

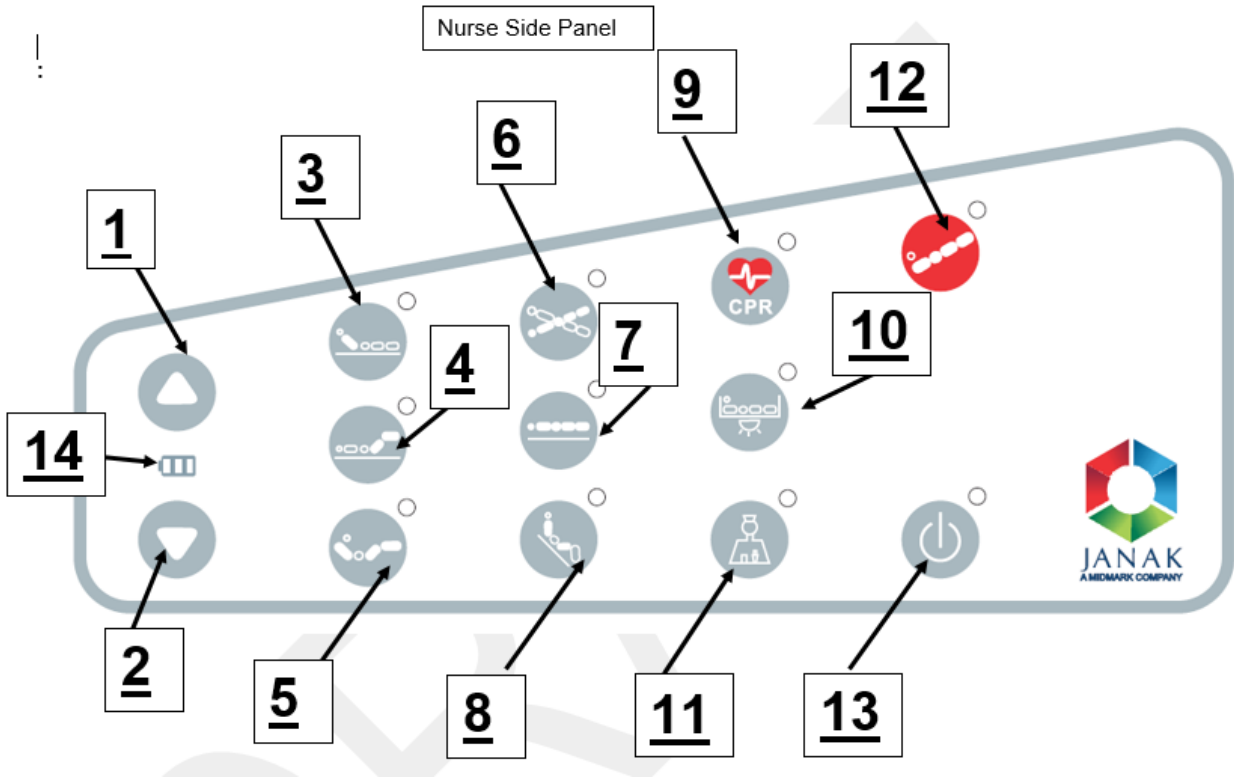

Patient Side Panel

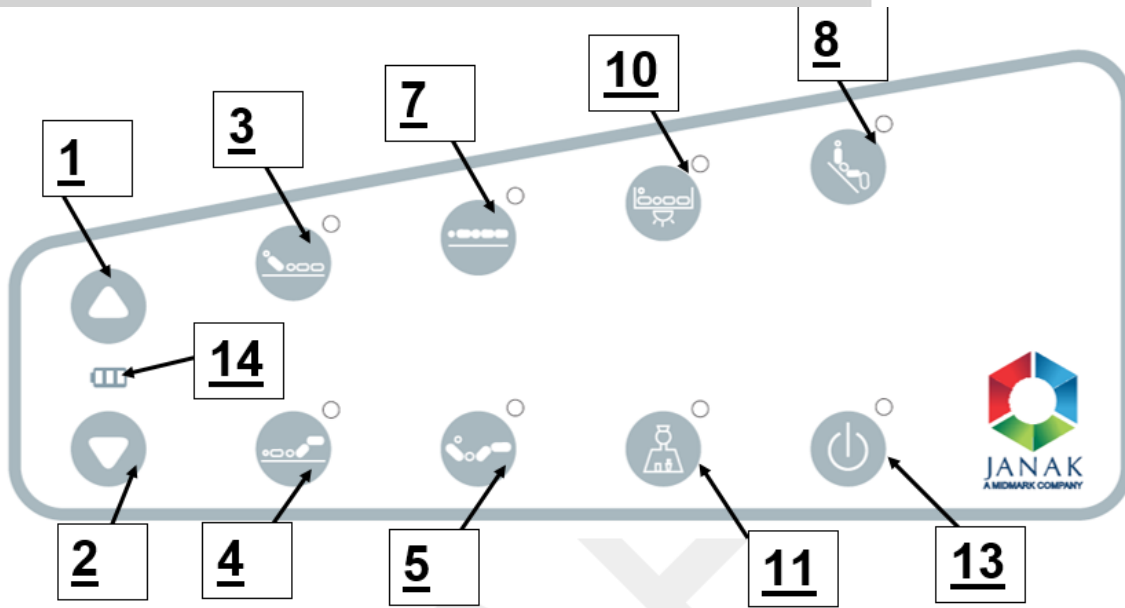

1. Press the button to raise back section, upper leg section, bed height, clockwise lateral tilt and sequential chair. First select the clinical position through embedded panels, then press 1 button.
2. Press the button to lower back section, upper leg section, bed height, anti-clockwise lateral tilt and sequential flat bed. First select the clinical position through embedded panels, then press 2 button.
3. Press to select the back section. Press 1 to raise and 2 to lower the back section.
4. Press to select the upper leg section. Press 1 to raise and 2 to lower the upper leg section.
5. Press to select auto contour position. Press 1 to raise back section and upper leg section simultaneously. Press 2 to lower the back section and upper leg section simultaneously.
6. Press to select TR/RTR position. Press 1 to lower the leg side (Reverse Trendelenburg). Press 2 to lower the back section (Trendelenburg).
7. Press to select Bed height function. Press 1 to raise the bed height. Press 2 to lower the bed height.
8. Press to select sequential chair function. Press 1 to attain a sequential chair position. Press 2 to flatten the bed sequentially.

9. Single continuous press the button will enable the bed to become flat and height of the bed goes to its lowest for nursing staff to perform emergency activities
10. Press to switch ON the under bed light. Press again to switch OFF the under bed light.
11. Press to activate nurse call. For nurse assistance call, the hospital should be equipped with nurse assistance facility compatible with bed.
12. Single continuous press the button to lower the head section of the bed for emergency cases.
13. Press the power button to put the system ON. With this button the system is alive. Press again the power button to put the system OFF. Once you OFF the system from side rail embedded system, HANDSET will also turn OFF.
14. It is battery level indicator. It shows the battery level remaining in the battery. Green color indicates that battery is full. Red color indicates that battery is drained.

Battery

The Bed is equipped with battery to allow emergency operation of the electric drive system. This ensures that all the power requirements can still be supplied in the event of power failure. The battery also has inbuilt fuse protection for the safety of battery as well as users.

Battery Indicators on Embedded Panels-

- Green indicator of power 'ON' button on embedded panels shows that the control elements are running on the main supply.
- Green indicator of battery symbol on embedded panels shows that the control elements are running on battery and battery is at full charge condition.
- Orange indicator of battery symbol indicates that control elements are running on the battery and battery is at medium charge condition.
- Red indicator indicates that the battery is fully discharged and no further function is possible on battery.



It is necessary to fully charge the battery at least once a month, especially when the Bed has not been in use for a long time. Otherwise the battery could get permanently discharged and/or damaged.



Always keep the Bed in charging condition for any emergency use.



Always keep the Bed connected to the main supply to get a fully charged battery.



Operations of the Bed on battery should be only as per its duty cycle (refer Electrical Specifications given above)



Batteries in the Bed may emit hydrogen gas. So do not bring fire or a heated object close to the Product.



With due consideration to environmental limitations, materials used to make the device are not affected by contact with any bio chemicals or bio waste. When used as per its intended purpose, all materials used in the product and accessories do not facilitate any breeding or growth of bio contaminants on their own.

MAINTENANCE & CLEANING

Maintenance

Each time before putting the Bed into service the user must check that –

- All the functions are in satisfactory working condition before the Bed is allotted to the next patient.
- Unplug the mains supply before carrying out any maintenance activity.
- Lubrication of all the mechanical joints / pins / moving parts / tightening of the bolts should be carried out at least once in three months, to avoid unnecessary sound and wear & tear of the Bed.
- At regular intervals carry out a visual inspection and check for any mechanical damage (e.g. loose /missing rivets, Screws, etc.)
- If any damage or malfunction is observed, immediately report this matter to the operator.
- If the Bed is not in use, it should be stored in a dry and flat indoor environment which is rodent free with gradient not more than 5 degrees.
- If the Bed is not in use for long term, it is necessary to conduct a test to check for functioning and any electrical or mechanical damage that may have been caused.
- Depending on the usage, certain moving parts may require replacement due to wear & tear. We recommend timely replacement with genuine Midmark spares for safety and trouble free operation.
- Lubricating the moving parts and tightening of the bolts should be carried out periodically.

Cleaning & Disinfecting –

- The Bed is not suitable for machine wash or steam cleaning.
- Do not use water jet for cleaning.
- Always disconnect the mains power cord from the electrical power supply before cleaning.
- Wipe the electrical components with dry cloth only. No traces of water or disinfectant should be present on any electrical component.
- The Bed cannot tolerate the influence of strong solvent, basic or alkaline liquids & caustic agents. Also, do not use concentrated insecticides on electrical parts.
- The cleaning agents must not contain any substances which bring about any change in the structure or adhesive property of the plastic.

- The Bed should be properly cleaned and disinfected to avoid infection and stains before allotment to anew patient.
- Immobilize the Bed & remove all the detachable accessories and components before cleaning. Any prolonged contact of urine or acid on mild steel can corrode the surface. Keep the surface dry at all times.
- Wearing suitable protective clothing, clean the surface with a disposable cloth soaked in a neutral detergent and hot water. We recommend use of approved disinfectant and cleaning agents to clean the Bed. E.g. Bacillocid or other similar agents.
- Start by cleaning the upper sections of the Bed & work along all horizontal surfaces. Work methodically towards the lower sections of the Bed. Take extra care to clean areas that may trap dust or dirt. All detachable accessories and its component can be cleaned separately.



Do not expose the Bed to excessive moisture as corrosion may cause personal injury or equipment damage.



Disposal:

Product and its defective parts especially elements made of plastic, steel, rubber and electrical /A electronics (including battery) material should be disposed off in accordance with the statutory rules and regulations prevailing for such disposal. The user is advised to contact 'Midmark India' for further assistance.

Routine Inspection Check List: Inspection By The User
 (Recommended Frequency: Every new allotment but not later than one month)

| Checks | Observations | OK | Not OK | Description of faults |
|---|---------------------------------------|----|--------|-----------------------|
| Visual check of the electrical components | | | | |
| ACP, embedded panel cables | Damage, loose, hanging | | | |
| Mains cable | Damage, loose, hanging | | | |
| Control elements, Attendant control Panel | Damage, loose, hanging | | | |
| Embedded Panels in Side rails | Damage, loose | | | |
| Visual check of the mechanical components | | | | |
| Main frame, base frame | Damage deformation | | | |
| Patient lifting pole, locating sleeves | Damage, bend | | | |
| Mattress | Damage | | | |
| Metal top | Damage deformation | | | |
| Side railing | Damage, position | | | |
| Railing dampeners (Gas springs) | Damage | | | |
| Counter balance springs for backrest | Loose | | | |
| Screws and rivets | Missing, loose | | | |
| Performance check of the electrical components | | | | |
| Embedded Panel | Function test | | | |
| Attendant control panel | Function test | | | |
| Control unit | Function test | | | |
| Motors | Function test | | | |
| Night lamps | Function test | | | |
| Performance check of all mechanical components | | | | |
| Detachable head and foot boards | Secure retention | | | |
| Castors | Locking & release | | | |
| Emergency lowering of the backrest (CPR handle) | Function test | | | |
| Side railing | Engagement, release, secure fastening | | | |
| Lower leg section | Function test | | | |

| | | | |
|-------------|----------------------------|--|---|
| Accessories | Fastening, damage, missing | | JANAK [®] Because we care. |
| Sign | Result of inspection | | Date |

Troubleshooting Guide

| PROBLEM | POSSIBLE CAUSES | REMEDY |
|--|---|--|
| Embedded panels system not functioning | <ul style="list-style-type: none"> • Main cables not plugged in • No main voltage at main supply/no current supply • Embedded panels cable not fixed properly to the control unit • Error in embedded panels unit | <ul style="list-style-type: none"> • Plug in the main cable • Check in the supply current • Fix in the embedded panels • Replace the embedded panels unit |
| Certain function not working | <ul style="list-style-type: none"> • Actuators not functioning • Control unit not functioning • ACP or embedded control panel not working • Cables of the actuators are loosened | <ul style="list-style-type: none"> • Replace the actuators • Replace or repair the control unit • Replace or repair the ACP/embedded panels. • Fix the cables firmly |
| Emergency operation not possible | <ul style="list-style-type: none"> • Battery rundown • Battery fuse blown off • Battery wire loosened | <ul style="list-style-type: none"> • Recharge the battery as recommended • Replace the fuse • Fix the battery wire firmly |
| Beeping sound from the control unit | <ul style="list-style-type: none"> • Overload on the bed • Internal or external fuse blown off • Battery rundown | <ul style="list-style-type: none"> • Reduce the load on the battery • Replace the fuse • Recharge the battery |
| No functions are operating | <ul style="list-style-type: none"> • Main cable loosened • Error in control unit | <ul style="list-style-type: none"> • Tighten the main cable firmly • Replace or repair the control unit |
| No function via embedded panels | <ul style="list-style-type: none"> • Embedded panels is defective | <ul style="list-style-type: none"> • Replace or repair the embedded panels |
| Manual emergency operation not functioning | <ul style="list-style-type: none"> • Bowden cables are set with too much slack or have become unhooked • Breaking in the Bowden cable • Squeaking in Bowden cable | <ul style="list-style-type: none"> • Readjust on release lever or adjust the wire by loosening the wire • Replace the Bowden cable • Lubricate the Bowden cable by oiling |

WARRANTY & SERVICE

Midmark India Pvt. Ltd. warrants solely to the end-users of this Stanchion Motorized bed that for a period specified in the warranty certificate, such Stanchion Motorized bed shall perform as per its intended use. The foregoing warranties shall not apply if the Product has been subjected to:(a) physical abuse, misuse, abnormal use, or use not consistent with Midmark India Pvt. Ltd.'s published directions, (b) any modification by anyone other than Midmark India Pvt. Ltd., or third parties designated by Midmark India Pvt. Ltd., or (c) fraud, tampering, negligence or accidents, depending upon the nature of the products.

Limitation Of Liability: In no event shall Midmark India Pvt. Ltd., be liable or obligated to the end-user in any manner for any special, non-compensatory, consequential, indirect, incidental, statutory or punitive damages of any kind. Including, without limitation. For injury or loss of life, lost profits lost sales. Lost revenue, loss of any software or data. or loss of use of any hardware, software or data, regardless of the form of action, whether in contract, tort, negligence, strict product liability, or otherwise, even if Midmark India Pvt. Ltd., has been informed of or is aware of the possibility of any such damages in advance. Midmark India Pvt. Ltd.'s total aggregate liability under this agreement and in relation to anything which Midmark India Pvt. Ltd., has done or not done in connection with this agreement shall be limited to the price of the products purchased and paid for by the end-user during the period in which such liability arises. The limitations set forth above shall be deemed to apply to the maximum extent permitted by applicable law and notwithstanding the failure of the essential purpose of any limited remedies.

In order to maintain functional safety and right to claim under the warranty only original 'Midmark' replacements parts may be used for ordering replacement parts, customer service request and further question, please contact our technical support department.

Disclaimer

Every effort has been made to ensure that the information in this user manual is valid. Midmark India Pvt. Ltd., reserves the right to change the information in this document without prior notice."

MANUFACTURER'S ADDRESS

Corporate Office:

Midmark India Pvt. Ltd.
Art Guild House, "A" Wing, Unit No.15 & 16, Lower Ground Floor, Phoenix Market City, LBS Road, Kurla(West), Mumbai, 400070
Toll Free: 1800 22 8020 | Tel: .:+91 22 4915 3000

Registered Office:

Midmark India Pvt. Ltd.
Janak House, Opp. IOC Depot, Shaikh Misry Road, Wadala (E),
Mumbai – 400037, India
[Tel: + 91 22 2413 0407](tel:+912224130407)

Toll-Free No.: 1800 22 8020 | Email: technicalsupport@midmark.com



MIDMARK INDIA PVT LTD.

Art Guild House, A-Wing, Unit No. 15 & 16, Phoenix Market City, LBS Road, Kurla (W),
Mumbai - 400 070 Toll Free: 1800 22 8020 | Tel: +91 22 4915 3000 | Fax: +91 22 4915
3100 www.midmark.in | india@midmark.com